

Simple  
and efficient  
needle stick  
PROTECTION



CONTA-GUARD™

# PROTECT YOURSELF against needle stick injury

CONTA-GUARD™ protects you against accidental needle sticks during the entire process – from preparation to disposal.

CONTA-GUARD™ laminated disposable shield seals the rear part of the needle. Once the contaminated needle has been safely sealed, all further handling is absolutely safe.

The CONTA-GUARD™ Stand permits safe one-handed de- and recapping of the needle, also when repeated injection to the same patient is needed.

The system can be used with all standard dental syringes and needles.

It is easy to handle and to integrate into the daily routines.





1

Remove the perforated field (pull-out opening) on the side/bottom of the box.



2

Remove the protective film. Press the needle cap through the hole in the CONTA-GUARD™, with the adhesive side facing up.



3

Place the prepared syringe in the stand. Then press the wings of the stand slightly. Hereby the needle cap is retained in the stand, so the syringe/needle can be lifted free of the needle cap.




4

The needle cap with CONTA-GUARD™ is left in the stand when injection takes place.



5

After injection: by one hand place the syringe in the stand to re-cap the needle. Be careful not to penetrate the needle cap with the sharp needle tip.

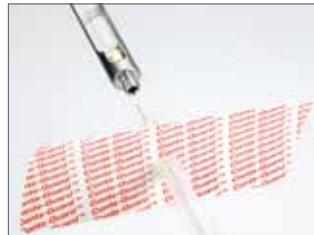

6

Lift up the syringe from the stand and unscrew the needle.



7

Fold the CONTA-GUARD™ firmly around the rear part of the needle.



8

The needle is now sealed and you are completely protected against needle stick injury.

View animated  
step by step-guide



# CONTA-GUARD™

An efficient protection against needle-stick injury. It protects during the entire process - from preparation to disposal of the contaminated needle.

## CONTA-GUARD™

Laminated disposable shields,  
box of 200 pcs., RØNVIG art. no. 2200  
Re-capping stand, RØNVIG art. no. 2222

Designed and manufactured in Denmark by



**RØNVIG** Dental Mfg. A/S

Gl. Vejlevej 59 · DK-8721 Daugaard · Tel.: +45 70 23 34 11  
Fax: +45 76 74 07 98 · E-mail: [export@ronvig.com](mailto:export@ronvig.com) · [www.ronvig.com](http://www.ronvig.com)