CTT™

Ceramic Tissue Trimmer

en da de fi fr nl no es it pl pt sv

INSTRUCTIONS FOR USE

DESCRIPTION	CTT™ is used under rotation in the air turbine handpiece (300,000 - 500,000 RPM). To be used <u>without</u> any water/spray cooling.
COMPATIBILITY	The CTT™ shank complies to EN ISO 1797:2017, type 3 and may only be used in turbine handpieces meeting the same standard.
INTENDED USE USER PROFILE	The CTT™ is intended for minor surgical procedures in dentistry. The use of CTT™ is restricted to educated dentists.
CHECKING THE INSTRUMENT	The product must be gently handled for protection of the tip. Before each use, make sure that the tip has not been damaged. If the tip is worn or damaged, dispose of the instrument.
PREPARATION	Mount the CTT™ into the turbine handpiece. Make sure that the CTT™ is well in place and safely fixed. The turbine coolant spray must be switched off.
APPLICATIONS:	1. CTT™ as a soft tissue trimmer: <ul style="list-style-type: none">Gingival modellingFreeing deep cavitiesLaceration of interradicular granulation tissuePapilectomy/removal of hyperplastic gingivaExposure of intraosseous implant sections and retinated teeth

Rinse away any blood and pus, and dry the area with a cotton sponge or with the air-syringe. The point of the CTT™ must be moved through the tissue with small "brushstroke" movements in a 30-45° degree, not forcing the point of the trimmer in the tissue or cutting too deep in only one cut.

2. CTT™ used for opening of sulcus, e.g. for impressions: Rinse away any blood and pus, and dry the area with a cotton sponge or with the air-syringe. The CTT™ is moved in the sulcus with an even slow sliding motion of the necessary depth. To avoid any tissue loss, it is important to preserve the epithelium of the marginal gingiva.

⚠ When using CTT™ - avoid excessive contact to tooth or bone tissue. Correctly used, the kinetic energy causes a heat-coagulating effect without "burning" the tissue. Using the CTT™ correctly, with all it's applications, you will find that the CTT™ is often able to replace not only electro-surgery and surgical blades, but in many cases also the use of retraction cord.

CLEANING	Clean in a dental washer disinfector using a universal program. Place the CTT™ in a burholder suited for dental washer and sterilizator.
STERILISATION	Do not use any solvent for cleaning purpose. The CTT™ is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.
STORAGE DISPOSAL	Keep in a dry place, protected against recontamination. After the last use, dispose of the CTT™ in a sharps container.
PRODUCT LIFETIME	3 years.
CAUTION Patent no	US Federal law restricts this device to sale by or on the order of a dental professional. US,368,336B1 . JP,2001-104319,A

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED. IFU-CTT-EN/DA/DE/FI/NL/FR/NO/ES/IT/PL/PT/SV-2019.11 VER 10