
# ASPIJECT®

## Active

en pt it es de da nl fr ro gr sk sv

REF 2070 | 2075 | 2080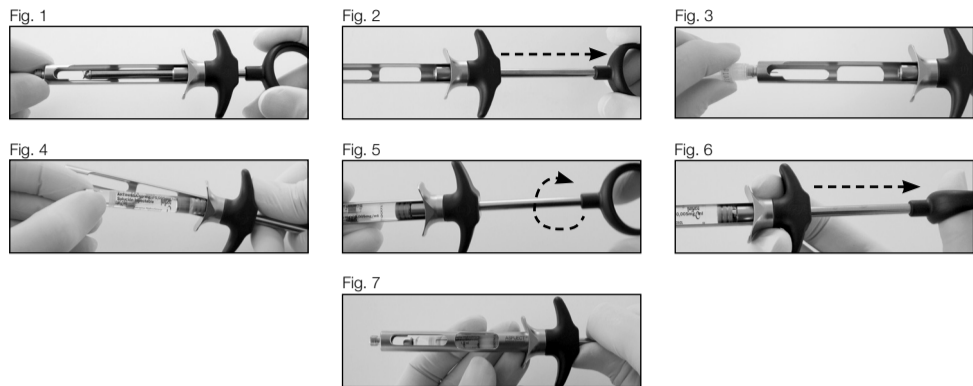

### INSTRUCTIONS FOR USE

**DESCRIPTION:** ASPIJECT® Active is a dental cartridge syringe for local anesthesia with active aspiration.  
**COMPATIBILITY:** ASPIJECT® Active is used with a standard dental needle and a standard dental cartridge according to the following:

REF	Needle thread	Cartridge
2070	M6	1.8 ml
2075	7/32"	1.8 ml
2080	7/32"	2.2 ml

**!** Use dental needles featuring plastic threads. Avoid using metal threads as they may cause deposit of metal shavings in the thread.

**INTENDED PURPOSE:** For infiltration and block anesthesia in the oral cavity.

**USER PROFILE:** To be operated by dental professionals authorized and trained to perform dental local injections.

**INSTRUCTIONS:**

- The spiral hook at the piston is extremely sharp, so to protect yourself, draw back the piston to let the spiral hook disappear into the rear part of the syringe, fig. 1.
- The spiral hook is now out of sight. Keep it there in order to avoid cutting injuries of your fingers, fig. 2.
- Screw on a standard dental needle, fig. 3.
- Draw back the plunger further, so that the whole spring-loaded retainer disappears into the rear of the syringe. Insert a cartridge with the rear end first, fig. 4.
- Engage the spiral hooks to the rubber bung of the cartridge by turning the plunger half a rotation clockwise, fig. 5.
- Active aspiration is achieved by pulling back the plunger, fig. 6.
- After the injection, screw off the needle and draw the plunger completely back. Remove the cartridge by letting it fall into your other hand, fig. 7.

**CLEANING:** Clean in a dental washer/disinfector using a universal program.

**STERILIZATION:** The ASPIJECT® Active is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (auto-clave) at 134°C/273°F for min. 3 minutes.

**STORAGE:** Keep in a dry place, protected against recontamination.

**DISPOSAL:** Dispose of the ASPIJECT® Active if the device is damaged or have visible signs of wear. After the last use, dispose of the ASPIJECT® Active in a recycling container in a clean condition.

**WARRANTY:** ASPIJECT® Active is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.