
ASPIJECT®

SIS

en de nl es ro da sv fr

REF 2085 | 2090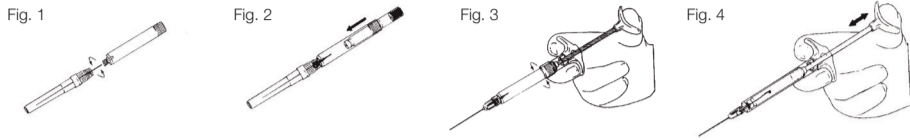

INSTRUCTIONS FOR USE

DESCRIPTION: ASPIJECT® SIS is a self-aspirating dental cartridge syringe for local anesthesia.
COMPATIBILITY: ASPIJECT® SIS is used with a standard dental needle and a standard dental cartridge according to the following:

REF	Needle thread	Cartridge
2085	M6	1.8 ml
2090	7/32"	1.8 ml / 2.2 ml

! Use dental needles featuring plastic threads.
 Avoid using metal threads as they may cause deposit of metal shavings in the thread.

INTENDED PURPOSE: For infiltration and block anesthesia in the oral cavity. Designed for use under sterile conditions.
USER PROFILE: To be operated by dental professionals authorized and trained to perform dental local injections.
INSTRUCTIONS:

1. Attach a needle. The non-sterile assistant screws on a standard dental needle while the sterile surgeon/assistant holds the barrel, fig. 1.
2. Insert a cartridge. The non-sterile assistant inserts a standard cartridge while the sterile surgeon/assistant holds the barrel, fig. 2.
3. Screw the barrel onto the sterile finger-grip. The sterile surgeon/assistant screws the loaded barrel onto the (sterilized) ASPIJECT® SIS finger-grip. The non-sterile assistant removes the needle cap, fig. 3.
4. Aspiration is accomplished by applying and easing a gentle pressure on the piston. The elastic recoil of the cartridge membrane – first pushed onto a hub at base of the barrel – creates the negative pressure in the cartridge, that secures distinct aspiration, fig. 4.

CLEANING: Clean ASPIJECT® SIS finger grip in a dental washer disinfectant using a universal program.
 Clean the cartridge barrel manually in tap water and a household dishwash soap. Rinse with water. Avoid cleaning the cartridge barrel in a dental washer disinfectant.

! Do not use any solvent for cleaning the barrel.

STERILIZATION: The ASPIJECT® SIS is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes. Keep barrel and finger-grip disassembled during sterilization (if syringe is sterilized assembled, the tensions caused by the thermal expansion of the plastic material might cause the barrel to break at the thread when the barrel is screwed off again).

BARRELS: ASPIJECT® SIS is supplied with one cartridge barrel. Additional barrels can be purchased in packs of 3.
STORAGE: Keep in a dry place, protected against recontamination.
DISPOSAL: Dispose of the ASPIJECT® SIS if the device is damaged or have visible signs of wear. After the last use, dispose of the ASPIJECT® SIS in a recycling container in a clean condition.
WARRANTY: The ASPIJECT® SIS is covered by a 5-year warranty. The barrel is not covered by the warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.
PRODUCT LIFETIME: 5 years.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.