CTT™

Ceramic Tissue Trimmer

INSTRUCTIONS FOR USE

DESCRIPTION:	CTT™ is used under rotation in the air turbine handpiece (300,000 - 500,000 RPM). To be used <u>without</u> any water/spray cooling.
COMPATIBILITY:	The CTT™ shank complies to ISO 14457:2017 and may only be used in turbine handpieces meeting the same standard.
INTENDED USE:	The CTT™ is intended for minor surgical procedures in dentistry.
USER PROFILE:	The use of CTT™ is restricted to educated dentists.
CHECKING THE INSTRUMENT:	The product must be gently handled for protection of the tip. Before each use, make sure that the tip has not been damaged. If the tip is worn or damaged, dispose of the instrument.
PREPARATION:	Mount the CTT™ into the turbine handpiece. Make sure that the CTT™ is well in place and safely fixed. The turbine coolant spray must be switched off.
APPLICATIONS:	<ol style="list-style-type: none"> CTT™ as a soft tissue trimmer: <ul style="list-style-type: none"> Gingival modelling Freeing deep cavities Laceration of interradicular granulation tissue Papilectomy/removal of hyperplastic gingiva Exposure of intraosseous implant sections and retinated teeth

Rinse away any blood and pus, and dry the area with a cotton sponge or with the air-syringe. The point of the CTT™ must be moved through the tissue with small "brushstroke" movements in a 30-45° degree, not forcing the point of the trimmer in the tissue or cutting too deep in only one cut.

2. CTT™ used for opening of sulcus, e.g. for impressions:
Rinse away any blood and pus, and dry the area with a cotton sponge or with the air-syringe. The CTT™ is moved in the sulcus with an even slow sliding motion of the necessary depth. To avoid any tissue loss, it is important to preserve the epithelium of the marginal gingiva.

⚠ When using CTT™ - avoid excessive contact to tooth or bone tissue. Correctly used, the kinetic energy causes a heat-coagulating effect without "burning" the tissue. Using the CTT™ correctly, with all it's applications, you will find that the CTT™ is often able to replace not only electro-surgery and surgical blades, but in many cases also the use of retraction cord.

CLEANING:	Clean in a dental washer disinfector using a universal program. Place the CTT™ in a burholder suited for dental washer and sterilizator. The CTT can also be cleaned in a manual way according to ISO 17664:2017, section 6.6.3, but this method has not been validated. Do not use any solvent for cleaning purpose.
STERILISATION:	The CTT™ is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.
STORAGE:	Keep in a dry place, protected against recontamination.
DISPOSAL:	After the last use, dispose of the CTT™ in a sharps container.
PRODUCT LIFETIME:	3 years.
CAUTION:	US Federal law restricts this device to sale by or on the order of a dental professional.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED. IFU_CTT_Multisprog_version11_2021.11