COMPO-JECT™

en INSTRUCTIONS FOR USE

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sv BRUKSANVISNING

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


no BRUKERVEILEDNING

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REF 20003

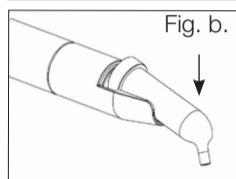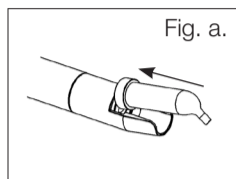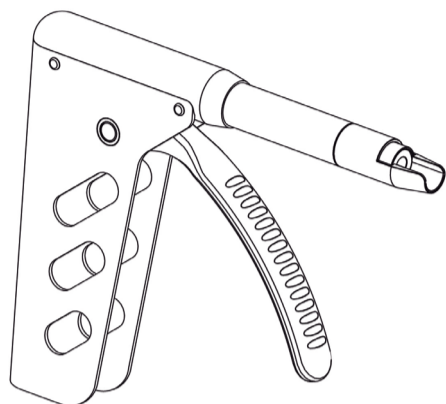INSTRUCTIONS FOR USE

en

INTENDED PURPOSE:	For delivery of dental materials in pre-filled unit-dose tips for direct oral application.
COMPATIBILITY:	COMPO-JECT™ is designed for pre-filled tips only.
MATERIALS:	COMPO-JECT™ is made from high grade, acid-resistant stainless steel.
USER PROFILE:	COMPO-JECT™ is intended for use in the field of dentistry and may be operated solely by dental professionals.
INSTRUCTIONS:	<ol style="list-style-type: none">1. Insert a unit-dose tip into the clip-in-lock at the front part of the applicator (Fig. a). Make sure that the tip is fully inserted.2. For easy placement of the material, the tip can be rotated in the required direction.3. Activate the lever at all times while using the applicator in the oral cavity. This will cause the piston to lock the unit-dose tip and thus prevent unintentional rotation or release of the tip.4. Release the tip from the clip-in-lock by pressing the front part of the tip downwards (Fig. b).
CLEANING:	Clean in a dental washer disinfectant using a universal program.
STERILIZATION:	The COMPO-JECT™ is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.
STORAGE:	Keep in a dry place, protected against recontamination.
DISPOSAL:	Dispose of the COMPO-JECT™ if the device is damaged or have visible signs of wear. After the last use, dispose of the COMPO-JECT™ in a recycling container in a clean condition.
WARRANTY:	COMPO-JECT™ is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.
PRODUCT LIFETIME:	5 years.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.