A-DAPT™

en da nl fr it de sv pl no es ro fi

REF 20210 | 20240 | 20245

INSTRUCTIONS FOR USE

DESCRIPTION:	The A-DAPT™ composite instruments w. silicone tips are designed for the adaption, modelling and contouring of composite and glass-ionomer restorations. Further silicone tips can be ordered from your A-DAPT™ dealer: REF 20215: 2 pcs. Silicone Tip, conical & rounded. REF 20220: 2 pcs. Silicone Tip, brush-shaped.
MATERIALS:	Handle: Acid resistant stainless steel. Tip: Silicone material and acid resistant stainless steel.
INTENDED PURPOSE:	An instrument for adapting, modelling and contouring of composite and glass-ionomer restorations.
USER PROFILE:	A-DAPT™ is intended for use in the field of dentistry and may be operated solely by dentists and dental technicians.
CLEANING:	After use, a greasy layer of resin may be left on the silicone tip. Such residues should be removed by wiping the tip with alcohol. Clean in a dental washer disinfectant using a universal program.
STERILIZATION:	The A-DAPT™ is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.
STORAGE:	Keep in a dry place, protected against recontamination.
DISPOSAL:	Dispose of the A-DAPT™ if the device is damaged or have visible signs of wear. After the last use, dispose of the A-DAPT™ in a recycling container in a clean condition.
WARRANTY:	A-DAPT™ is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty. The tips are not included in the warranty.
PRODUCT LIFETIME:	5 years.

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© RÖNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.
Dental Mfg. A/S