DENTO-PREP™ DUST-CABINET 12V INSTRUCTIONS FOR USE





Intended use:

DUST-CABINET is intended for powder blasting procedures with DENTO-PREP™.

Preparation:

Attach the power supply cable and your DUST-CABINET is ready for use. The window in the DUST-CABINET can be removed by pushing/sliding it upwards.

Safety precautions:



Use only the power supply delivered with the device.



Disconnect the power supply from DUST-CABINET during cleaning and filter change.



Gloves should be worn whenever using DUST-CABINET.

Maintenance:

Regular cleaning of the DUST-CABINET is recommended: Remove the window and vacuum the used powder.

Filter change:

The DUST-CABINET is equipped with a dust filter which protects the built-in electronic suction system against the powder. Inspect the filter regularly. If the filter is damaged or worn replace the filter by following step 1-4.







DENTO-PREP



Technical information:

Dimensions: H 180 - W 222 - D 298

Weight: 3776 grams Power: 12 VDC / 1A

LED HF210034, 12 VDC, 3,9W, 4000 K, 900 lumen Lamp:

Cabinet Material: Powder coated steel

Window: Plexiglass

Disposal:

Dispose of the DUST-CABINET if the device is damaged or have visible signs of wear.

After the last use, dispose of the DUST-CABINET in a separate collection of electronic waste according to WEEE Directive 2012/19/EU.

Warranty: 2 years



