

WASH TRAY™ Instructions for Use

REF 30005 1/1 Tray (275 x 180mm) for max. 20 instruments.
REF 30006 ½ Tray (131 x 180mm) for max 9 instruments.
REF 30007 ¼ Tray (73 x 180mm) for max 5 instruments.

Intended Use	For the fixation and organization of dental instruments during thermo-disinfection, sterilization, storage and chairside use
Spare parts available	REF 29995 Silicone Strips Set for 1/1 Tray 30005 REF 29997 Silicone Strips Set for ½ Tray 30006 REF 29999 Silicone Strips Set for ¼ Tray 30007
First use	WASH TRAY™ is delivered non-sterile and must be cleaned and sterilized before the first use.
Limitations on reprocessing	Repeated reprocessing has minimal effect on WASH TRAY™
Containment and transportation	No particular requirements
Preparation for cleaning	No particular requirements
Cleaning: Automated	Clean in a dental thermal disinfectant by using a universal washing program 90°C 5 minutes. Drying: No particular requirements. Standard dental thermal disinfectant agents may be used such as: a) Detergent, neodisher MediClean forte by Dr. Weigert, 30 ml/cycle b) Neutralizer, neodisher Z by Dr. Weigert, 9 ml/cycle c) Rinse aid, neodisher MediKlar by Dr. Weigert, 2,5 ml/cycle d) Dishwasher salt, Neophos coarse salt
Cleaning: Manual	Not applicable
Sterilization	Sterilize in a pre-vacuum (ISO 13060 Class B) steam sterilization autoclave at standard program 134° C, 2 bar, 5,5 min, operation time 25 min. Drying for 25 min.
Storage	Keep in a dry place, protected against contamination.
Inspection and functional testing	No particular requirements.
Additional information	When cleaning and sterilizing multiple instruments in one automated cleaning and sterilization cycle, ensure that the maximum loads of the thermosinfectant and sterilizer have not been exceeded.
Disposal	Dispose of the WASH TRAY™ if the device is damaged or have visible signs of wear. After the last use, dispose of the WASH-TRAY™ in a recycling container in a clean condition.

File: IFU_WASH TRAY_EN_version 02_2021.11



RØNVIG Dental Mfg. A/S

Gl. Vejlevej 59 • DK-8721 Dagaard • Denmark
Tel.: +45 70 23 34 11 • Fax: +45 76 74 07 98 • E-mail: export@ronvig.com www.ronvig.com

