

Using the Installation Kit

Contents: Hose, tee-fitting, female quick connector and male connector.

- REF** 1925 Installation Kit for Ø 5 mm hose
- REF** 1926 Installation Kit for Ø 6 mm hose
- REF** 1928 Installation Kit for Ø 8 mm hose
- REF** 1929 Installation Kit for Ø 10 mm hose

Intended Use


Optional accessory for DENTO-PREP™ to create direct and permanent access to compressed air. To be mounted by a qualified dental service technician.

Mounting Instruction

Locate the air pressure supply close to the desired installation outlet. Turn off the air. Cut off your air supply hose and fit the ends into the side openings of the tee without using tools. Attach the DENTO-PREP™ hose as shown.

The operating pressure range of DENTO-PREP™ must be set between 2-8 bar (30-120 psi) dependent on the desired application. Therefore, an air pressure reducer should be mounted between the T-fitting and the female quick connector.

Make sure that the main air line is fitted with a filter and water trap. Clean and dry air is essential to avoid clogging of the DENTO-PREP™ device.

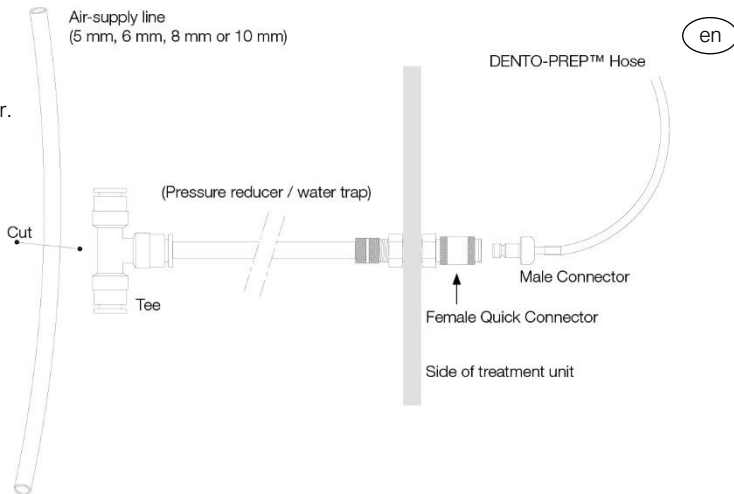

RØNVIG Dental Mfg. A/S • Gl. Vejlevej 59 • DK-8721, Daugaard • Denmark
Tel.: +45 70 23 34 11 • Fax: +45 76 74 07 98 • E-mail: export@ronvig.com • www.ronvig.com

