
PAROJECT®

en da no sv fi fr it es de nl pt sk ro cs it

REF 2013 | 2016 | 2018 | 2030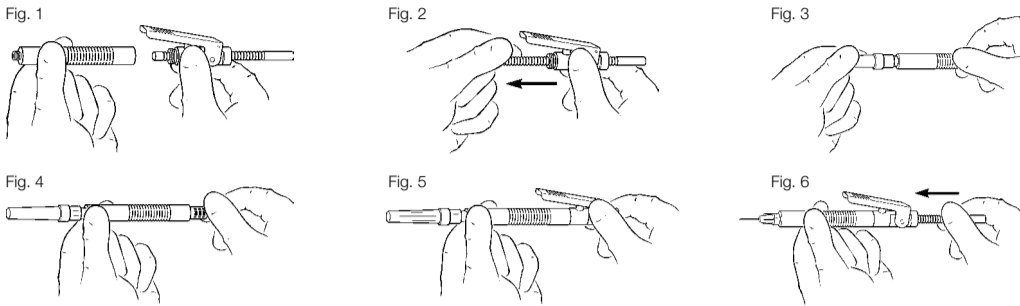

INSTRUCTIONS FOR USE

DESCRIPTION: PAROJECT® is a compact syringe with a controlled small dosage permitting a slow non-traumatic injection.
COMPATIBILITY: PAROJECT® is used with a standard dental needle and a standard dental cartridge according to the following:

REF	Needle thread	Cartridge
2013	M6	1.8 ml
2016, 2018	7/32"	1.8 ml
2030	7/32"	2.2 ml

Use dental needles featuring plastic threads. Avoid using metal threads as they may cause deposit of metal shavings in the thread.

INTENDED PURPOSE: Dental syringe for intraligamental local analgesia.
USER PROFILE: To be operated by dental professionals authorized and trained to perform dental local injections.

- INSTRUCTIONS:**
1. Disassemble the syringe into two parts: front (barrel) and rear (lever housing), fig. 1.
 2. Remove the plunger by pulling the plunger through the lever housing, fig. 2.
 3. Screw on the disposable dental needle, fig. 3.
 4. Insert a standard dental cartridge into the barrel, fig. 4.
 5. Reassemble the syringe by screwing the barrel onto the lever housing, fig. 5.
 6. Insert the plunger into the lever housing and push it until contact is made with the cartridge, fig. 6.

The plunger can only be inserted from the rear of the lever housing and with the smallest smooth part of the plunger going into the lever housing first.

The PAROJECT® is now loaded and ready for use.

CLEANING: Clean in a dental washer disinfectant using a universal program.
STERILIZATION: The PAROJECT® is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.
STORAGE: Keep in a dry place, protected against recontamination.
DISPOSAL: Dispose of the PAROJECT® if the device is damaged or have visible signs of wear. After the last use, dispose of the PAROJECT® in a recycling container in a clean state.
WARRANTY: PAROJECT® is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.
PRODUCT LIFETIME: 5 years.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED. IFU_PAROJECT_multisprog_version12_2023.11