ASPIJECT® PLUS


Gl. Vejlevej 59 · DK-8721 Daugaard · Denmark
Tel.: +45 70 23 34 11 · Fax: +45 76 74 07 98 · E-mail: contact@ronvig.com · www.ronvig.com

© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.
IFU_ASPIJECT_PLUS_multisprog_version01_2023.10

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REF 2019 | 2020 | 2021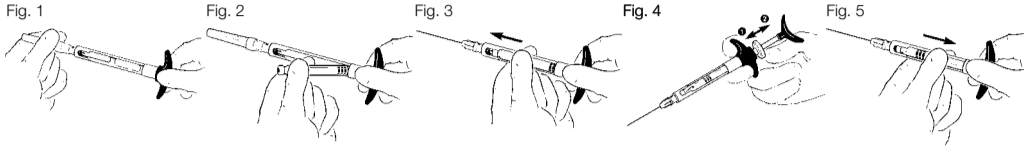

INSTRUCTIONS FOR USE

DESCRIPTION: ASPIJECT® PLUS is a self-aspirating dental cartridge syringe for local anesthesia.
COMPATIBILITY: ASPIJECT® PLUS is used with a standard dental needle and a standard dental cartridge according to the following:

REF	Needle thread	Cartridge
2019	M6	1.8 ml
2020	7/32"	1.8 ml
2021	7/32"	2.2 ml

Use dental needles featuring plastic threads.
Avoid using metal threads as they may cause deposit of metal shavings in the thread.

INTENDED PURPOSE: For infiltration and block anesthesia in the oral cavity.

USER PROFILE: To be operated by dental professionals authorized and trained to perform dental local injections.

INSTRUCTIONS:

1. First screw a standard dental needle on to the syringe, fig. 1.
2. Insert the cartridge by the rear of the cartridge first, fig. 2.
3. Push the cartridge forward to engage it with the needle, fig. 3.
4. Passive aspiration: Gently push and immediately after release the pressure on the aspiration disc.
The distortion of the cartridge rubber will cause a negative pressure inside the cartridge needed for aspiration.
5. After the injection, push the cartridge backwards to disengage it from the needle. Remove the cartridge, fig. 5.

CLEANING: Clean in a dental washer/disinfector using a universal program.

STERILIZATION: The ASPIJECT® PLUS is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (auto-clave) at 134°C/273°F for min. 3 minutes.

STORAGE: Keep in a dry place, protected against recontamination.

DISPOSAL: Dispose of the ASPIJECT® PLUS if the device is damaged or have visible signs of wear. After the last use, dispose of the ASPIJECT® PLUS in a recycling container in a clean condition.

WARRANTY: ASPIJECT® PLUS is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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