ASPIJECT®


Active Compact

en da no sv fi fr it es de nl lt sk

REF

2045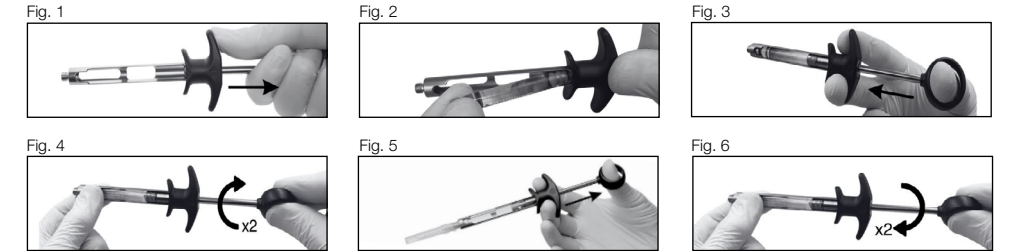

INSTRUCTIONS FOR USE

DESCRIPTION:

ASPIJECT® Active Compact is a dental cartridge syringe for local anesthesia with active aspiration.

COMPATIBILITY:

ASPIJECT® Active Compact is used with a standard dental needle and a standard dental cartridge according to the following:

| REF  | Needle thread | Cartridge                             |
|------|---------------|---------------------------------------|
| 2045 | M6            | 1.8 ml (with recessed rubber stopper) |

Use dental needles with plastic threads.  
Avoid using dental needles with metal threads, as they may deposit metal shavings in the syringe thread.

INTENDED PURPOSE:

For infiltration and block anesthesia in the oral cavity.

USER PROFILE:

To be operated by dental professionals authorized and trained to perform dental local injections.

INSTRUCTIONS:

- Retract piston rod fully, fig. 1.
- Insert a cartridge with the rubber plunger first, fig. 2.
- Hold the cartridge firmly in place. Push the piston rod forward to attach the threaded plunger tip to the rubber plunger, fig. 3.
- Engage the threaded plunger tip by turning the plunger two half rotations clockwise, fig. 4.
- Active aspiration is achieved by pulling back on the thumb ring, fig. 5.
- After the injection, remove the needle and rotate the plunger two half rotations counterclockwise. Draw the piston rod completely back. Tilt syringe to let the cartridge fall into your other hand, fig. 6.

CLEANING:

Clean in a dental washer disinfecter using a universal program.

STERILIZATION:

The ASPIJECT® Active Compact is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.

STORAGE:

Keep in a dry place, protected against recontamination.

DISPOSAL:

Dispose of the ASPIJECT® Active Compact if the device is damaged or have visible signs of wear. After the last use, dispose of the ASPIJECT® Active Compact in a recycling container in a clean condition.

WARRANTY:

ASPIJECT® Active Compact is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.