
ASPIJECT®

Active Compact

en da no sv fi fr it es de nl lt sk

REF

2040 | 2041 | 2042 | 2043 | 2044

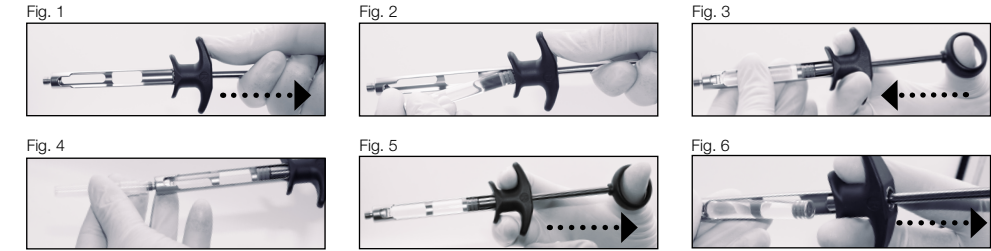INSTRUCTIONS FOR USE

DESCRIPTION:

ASPIJECT® Active Compact is a dental cartridge syringe for local anesthesia with active aspiration.

COMPATIBILITY:

ASPIJECT® Active Compact is used with a standard dental needle and a dental cartridge according to the following:

REF	Needle thread	Cartridge
2040	M6	1.8 ml standard
2041	7/32"	1.8 ml standard
2042	7/32"	2.2 ml standard
2043	7/32"	1.8 ml dentsply self-aspirating
2044	7/32"	2.2 ml dentsply self-aspirating

⚠

Use dental needles with plastic threads. Avoid using dental needles with metal threads, as they may deposit metal shavings in the syringe thread.

INTENDED PURPOSE:

For infiltration and block anesthesia in the oral cavity.

USER PROFILE:

To be operated by dental professionals authorized and trained to perform dental local injections.

INSTRUCTIONS:

1. Retract piston rod fully, fig. 1.

2. Insert a cartridge with the rubber plunger first, fig. 2.

3. Hold the cartridge firmly in place. Push the piston rod forward to attach the harpoon to the rubber plunger, fig. 3.

4. Screw on a standard dental needle, fig. 4.

5. Active aspiration is achieved by pulling back on the thumb ring, fig. 5.

6. After the injection, screw off the needle and draw the piston rod completely back. Tilt syringe to let the cartridge fall into your other hand, fig. 6.

CLEANING:

Clean in a dental washer disinfector using a universal program.

STERILIZATION:

The ASPIJECT® Active Compact is delivered non-sterile and should be sterilized before first use. Sterilize in moist heat (autoclave) at 134°C/273°F for min. 3 minutes.

STORAGE:

Keep in a dry place, protected against recontamination.

DISPOSAL:

Dispose of the ASPIJECT® Active Compact if the device is damaged or have visible signs of wear. After the last use, dispose of the ASPIJECT® Active Compact in a recycling container in a clean condition.

WARRANTY:

ASPIJECT® Active Compact is covered by a 5-year warranty. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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© RØNVIG Dental Mfg. A/S. ALL RIGHTS RESERVED.